


## Velox Imaging Local Agent Step by Step Installation Instruction


**Step 1:** Download the installer [here](#).

**Step 2:** Run **LocalAgentInstaller.exe**


**Step 3:** Press Install button:

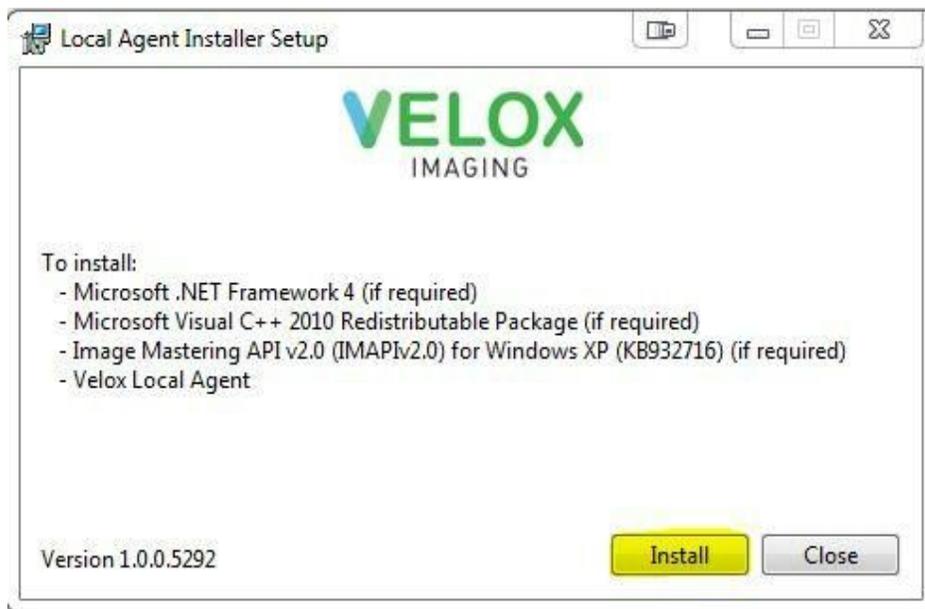

**Step 4:** Insert User Description: Clinic Name and Computer Name (for example, *LA Cardiology\_Reception 2*) and press Register button:

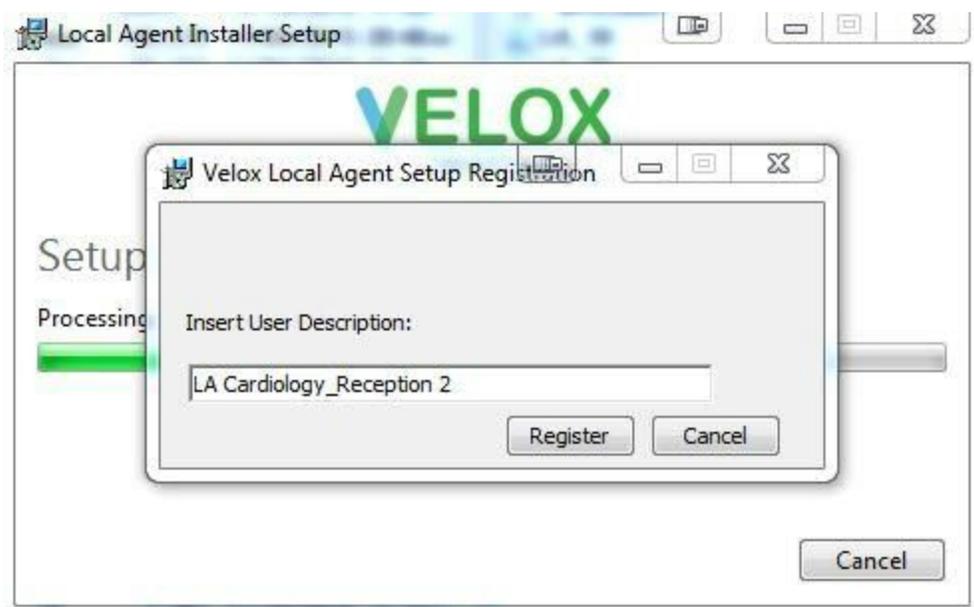


**Step 5:** Press Next button:

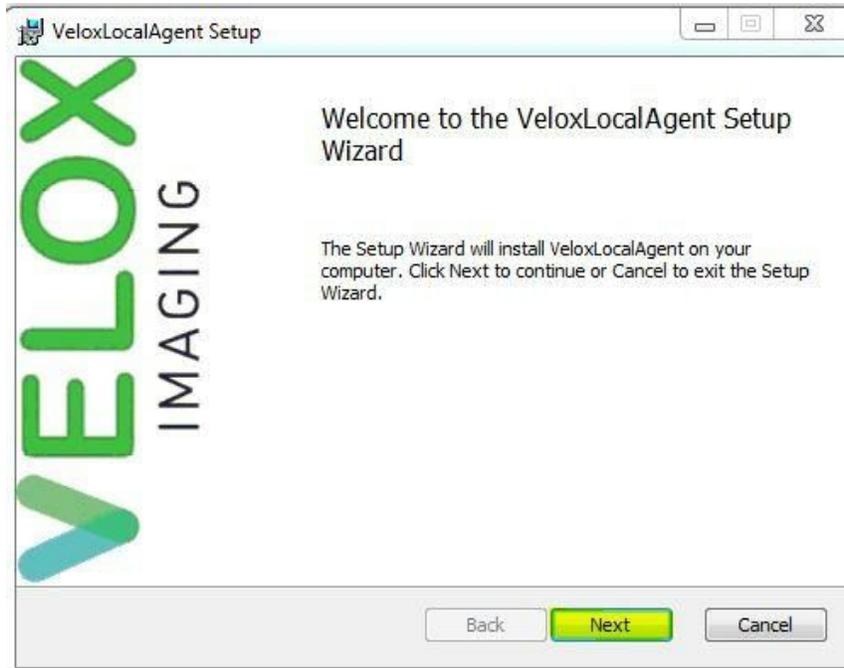


**Step 6:** Accept the License Agreement and press Next:




**Step 7:** Click Next button to install to the default folder or click Change button to choose another destination.

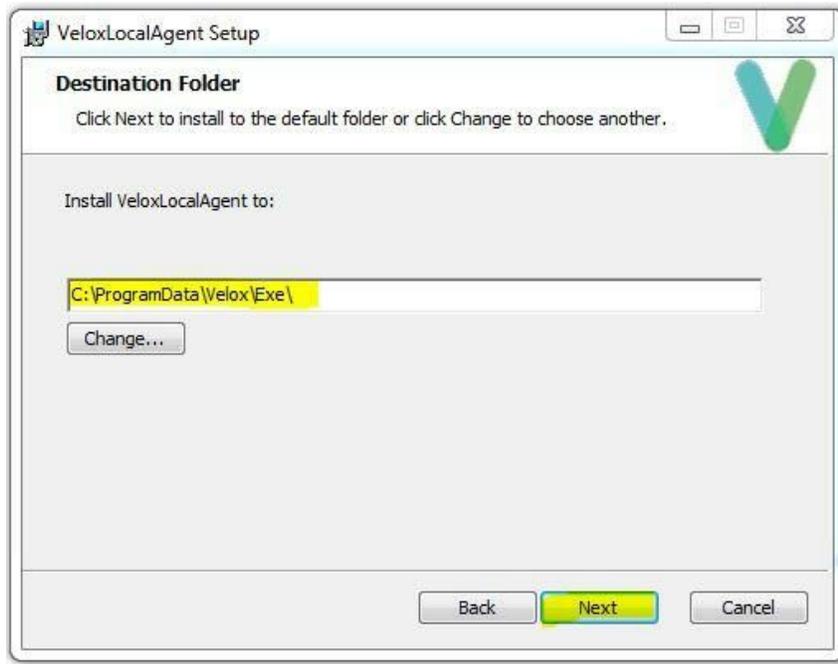


**Step 8:** Click Install button to begin the installation of Velox Imaging Local Agent.

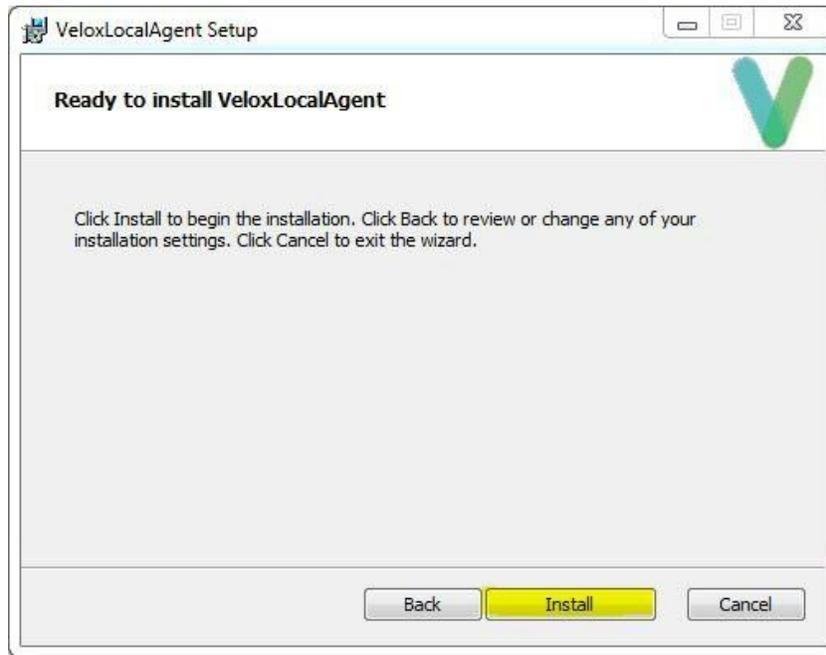

**Step 9:** Click Finish button to end installation and exit the Setup Wizard.

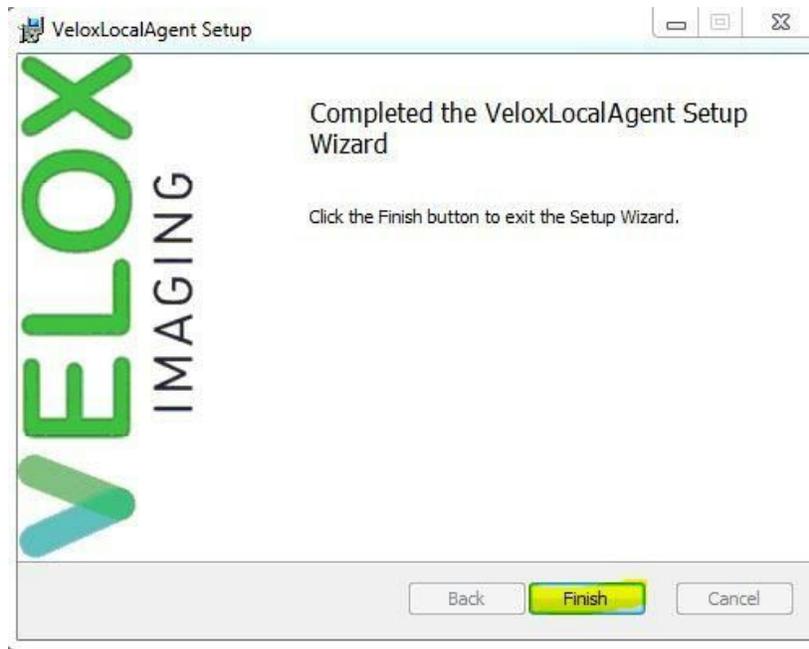

**Step 10:** Velox Imaging Local Agent has been installed on your computer. Press Close button:

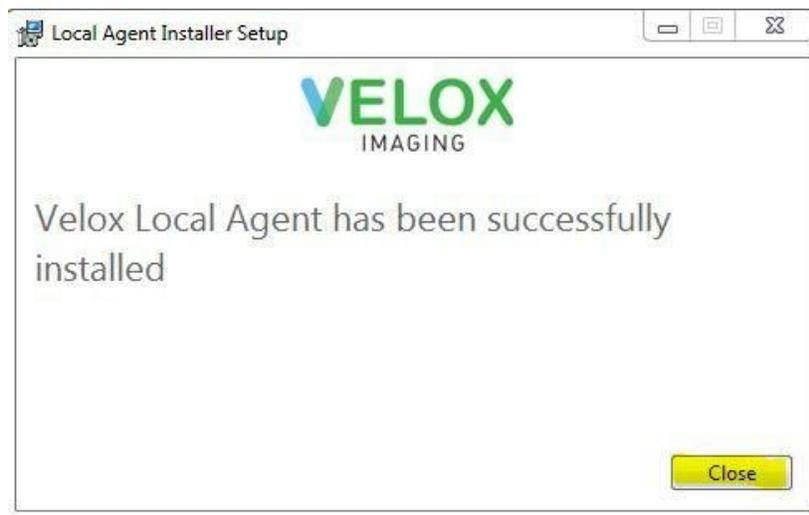

Once the program was installed and started, the **Velox Imaging Local Agent Icon** should appear in the system tray of your operating system:

